


Dorsal Night Splint

Designed to help provide support while sleeping for patients with Plantar Fasciitis and Achilles Tendonitis

 **Warning:** Remove before walking. Walking while wearing the Dorsal Night Splint could result in slipping and falling.



1. Remove the Dorsal Night Splint from bag and open hook closures.



2. Slide foot and heel into elastic piece and tighten the ankle straps and the toe straps. Re-adjust if necessary for comfort.

3. Wrap the calf strap around calf, and adjust for comfort if necessary.



Hand wash cold water. Air dry.




Manufactured For:

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com

Dorsal Night Splint

Designed to help provide support while sleeping for patients with Plantar Fasciitis and Achilles Tendonitis

 **Warning:** Remove before walking. Walking while wearing the Dorsal Night Splint could result in slipping and falling.




1. Remove the Dorsal Night Splint from bag and open hook closures.



2. Slide foot and heel into elastic piece and tighten the ankle straps and the toe straps. Re-adjust if necessary for comfort.

3. Wrap the calf strap around calf, and adjust for comfort if necessary.

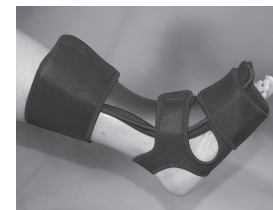

Hand wash cold water. Air dry.



Manufactured For:

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com