

I N S T R U C T I O N S F O R U S E

Ossur North America
 27412 Aliso Viejo Parkway
 Aliso Viejo, CA 92656, USA
 Tel: +1 800 233 6263
 Tel: +1 949 362 3883
 Fax: +1 949 362 3888
 ossurusa@ossur.com

Ossur - Braces and Support
 742 Pancho Road
 Camarillo, CA 93012, USA
 Tel +1 805 484 2600
 Fax +1 800 284 4356

Ossur - Custom Braces
 19762 Pauling
 Foothill Ranch, CA 92610, USA
 Tel +1 949 859 4407
 Fax +1 800 453 4567

Ossur Spinal
 Mid Atlantic Corporate Center
 1414 Metropolitan Ave.
 P.O. Box 566
 Thorofare, New Jersey 08086, USA
 Tel: +1 856 848 0525
 Fax: +1 856 845 1148
 spine@ossur.com

Ossur Canada
 12111 Jacobson Way
 Richmond, BC
 V6W 1L5, Canada
 Tel: +1 604 241 8152
 Fax: +1 604 241 8153

Ossur Europe
 P.O. Box 120
 5690 AC Son en Breugel
 The Netherlands
 Tel: 00800 3539 3668
 Tel: +31 499 462840
 Fax: +31 499 462841
 saleseurope@ossur.com

Ossur Europe
 Kundenservice Deutschland
 Römerfeldstraße 2
 50259 Pulheim, Germany
 Tel: +49 (0) 2238 30 58 50
 Fax: +49 (0) 2238 30 58 01
 info-deutschland@ossur.com

Ossur Nordic
 Salagatan 16a
 P.O. Box 67
 751 03 Uppsala, Sweden
 Tel: +46 1818 2200
 Fax: +46 1818 2218
 info@ossur.com

Ossur International Markets
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel: +354 515 1300
 Fax: +354 515 1370
 international@ossur.com

Ossur Asia Pacific
 2 Redbank Road
 Northmead NSW 2152
 Australia
 Tel +61-2-9630-9206
 Fax +61-2-9630-9268
 info-asiapacific@ossur.com



Exoform® Carpal Tunnel Wrist Brace

Left Part #	Size	Wrist	Palm	U/M
517083	Small	5.25" to 6.25"	6.75" to 8"	1 ea
517085	Medium	6.25" to 7.75"	8" to 9.25"	1 ea
517087	Large	7.75" to 10"	9.25" to 10.5	1 ea
Right Part #≈	Size	Wrist	Palm	U/M
517073	Small	5.25" to 6.25"	6.75" to 8"	1 ea
517075	Medium	6.25" to 7.75"	8" to 9.25"	1 ea
517077	Large	7.75" to 10"	9.25" to 10.5	1 ea

Ossur Head Office
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel + 354 515 1300
 Fax +354 515 1366
 mail@ossur.com



**EXOFORM® CARPAL
 TUNNEL WRIST BRACE**



Life Without Limitations

Exoform® Carpal Tunnel Wrist Brace

USES

Tendonitis, Carpal Tunnel Syndrome, arthritis and other selected injuries

CAUTION

Use this product only under the guidance of a physician or certified medical professional. If any problems occur with the use of this product, please immediately contact your medical professional. Do not over-tighten straps.

PHOTO CAPTIONS

- A. Straps with adjustable clips
- B. Thumb hole
- C. Contact closure sizing strap
- D. Web space
- E. Strap hooks

FITTING INSTRUCTIONS


1. Unfasten straps and place hand into brace so the thumb passes through the thumb hole and the web space contacts the skin between the thumb and index finger. The strap hooks should be on top of the arm.
2. Fasten straps by placing the square hole in each clip over the corresponding hook.
3. The top portion of the brace should be in-line with your forearm. If not, adjust the contact closure sizing strap until it is in-line with your forearm.
4. Pull on the end of each strap until comfortably tight. Do not over-tighten straps. The web space should just make contact with the web between the thumb and index finger.

5. To loosen the straps, lift up on the tab located on each clip until the tension is released. To remove the product, pull each clip toward you to disengage from the hooks.


WASHING INSTRUCTIONS

This product may be cleaned using a mild detergent and cold water. AIR DRY ONLY. The padding can be removed for hand washing. Do not use bleach or strong detergents. Do not wash in washing machine or place in dryer. Care should be taken to replace the padding in its original location.

Exoform® Carpal Tunnel Wrist is covered by one or more of the following Patent Nos. 5,334,135; 5,695,452; 5,713,837. Other U.S. and foreign patents pending. ® indicates registration in U.S.A. and selected countries only.

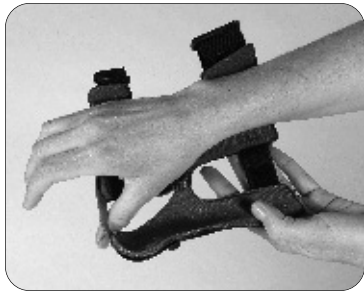

1

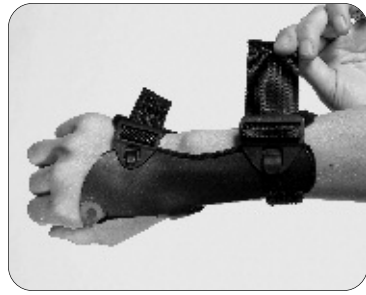


4




2




5

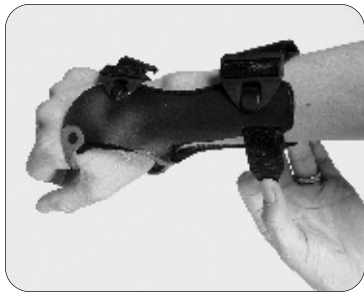

3

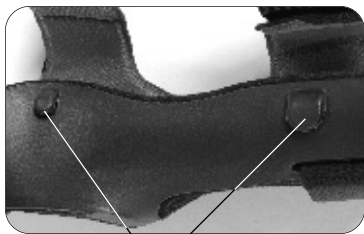

E.

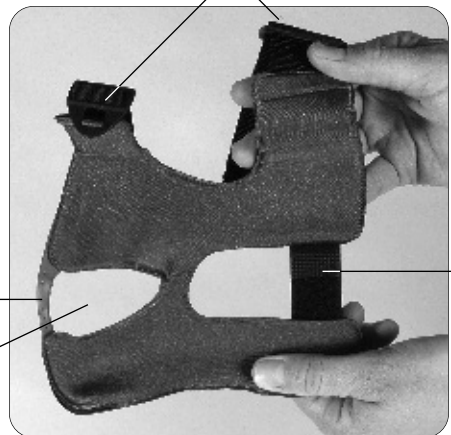

D.

B.

A.

C.