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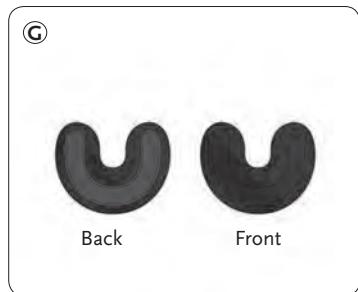
Form Fit® Knee Brace

CAUTION:

Before application, confirm all product components are included and undamaged. If any parts are missing or damaged, please contact your local Ossur Customer Service Representative. Contact information can be found on the back of this document.

Components: (Figure 1)

- A. Softgood (sleeve or wrap; Form Fit sleeve is shown)
- B. Patella opening
- C. Hinge pockets (2)
- D. Straps (2)
- E. Hinges (2)
- F. Hinge arms (4)
- G. Form Fit patella buttress (sleeve version only)

Indications: Mild to moderate sprains and strains of the knee

Application:

NOTE: For best results: i) When contouring hinge arms, remove hinges from hinge pockets and bend hinge arms as desired. Re-apply hinges to softgood before application ensuring the hinge arms are completely seated within the hinge pockets and the hinge condyle is secured to the softgood.; ii) Adjust patella as necessary before application. It can be trimmed or removed as appropriate.

1. Application of Brace:

- a. **Sleeve version:** Loosen straps and pull softgood up over the knee so that the patella opening is centered over the patella. (Figure 2)
 - b. **Wrap-around version:** Unfasten straps and place the open softgood behind the knee. Wrap the softgood around the leg (lower portion first) and fasten firmly with the contact closures. The patella opening should be centered over the patella. (Figure 3)
2. **Fasten Straps:** Fasten the straps in the order shown. The order is the same for both sleeve and wrap-around versions. (Figure 4)

Washing Instructions:

1. Remove hinges and close hook/loop fasteners before washing.
2. Hand wash using 30°C (86°F) soapy water. Rinse thoroughly and line dry only.
3. Do not machine wash or dry.
4. For best results, rinse after every use.

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